

Oxivir[®] Excel[™]

Broad Spectrum Cleaner & Disinfectant for Nonporous Hard Surfaces

Description

Diversey Care Oxivir Excel is a highly concentrated one step cleaner disinfectant with broad spectrum efficacy and short contact times. Based on Diversey Care patented Accelerated Hydrogen Peroxide formulation Oxivir Excel can be used on hard, nonporous surfaces and some soft furnishings.

Key properties

- Bactericidal, Virucidal, Yeasticidal
- Disinfects in as little as 1 minute
- Positive user and surface safety profile
- Convenient , no preparation or laundry required
- Effective Cleaner
- Fragrance free
- Active breaks down into water and oxygen
- No rinsing required

Benefits

- Broad efficacy provides the highest levels of patient protection. Extensively tested against European norms and effective against multiple pathogens of concern, carries a full virucidal claim including *Poliovirus*, *Adenovirus*, and *Murine norovirus*, considered effective against all enveloped viruses such as: HIV, HBV, HCV, *Influenza*, *Vaccinia virus* and *Coronavirus*.
- Short contact times keep the surface wet thus ensuring disinfectant efficacy
- Excellent cleaning performance, including descaling and lime limitation removes the need for a pre-cleaning step
- Excellent user safety profile reduces worker risk, with no hazard warnings; does not contain respiratory irritants
- AHP formulation is gentle on surfaces extending equipment life
- Suitable for use in high risk areas in Healthcare, including OR, ICU, NICU and patient rooms

Use instructions

Dilution

To achieve disinfection claims made follow the recommended dosing.



Application

Spray cleaning and disinfection:

1. Remove gross soils.
2. Apply to surfaces by either spraying the wiping cloth and wiping surfaces or applying directly to the surface through use of a trigger sprayer.
3. After the contact time*, either wipe dry or allow to air dry.

Bucket cleaning and disinfection:

1. Remove gross soils.
2. Apply to surfaces with mop/cloth.
3. After the contact time*, either wipe dry or allow to air dry.
4. Change the solution if it becomes visibly soiled or at least every 24 hours

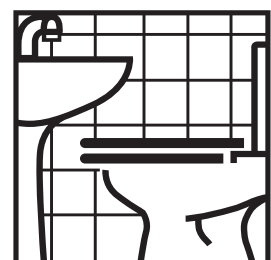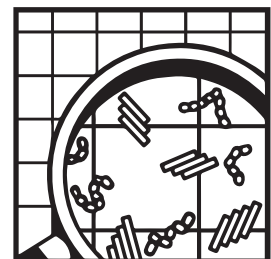

© A.I.S.E.

Oxivir[®] Excel[™]

Broad Spectrum Cleaner & Disinfectant for Nonporous Hard Surfaces

Dry Wipes Disposable Wiping System

1. Prepare Dry Wipes Disposable Wiping System according to instructions.
2. Remove gross soils.
3. Remove single wipe and wipe the surface.
4. After the contact time**, either wipe dry or allow to air dry.
5. Change the wipe when visibly soiled or moving to other cleaning area

Important: Do not use warm water. Do not mix with other products. Do not use on surfaces sensitive to water or acid.

* Ensure surface stays wet for duration of contact time

Technical data

| | |
|---------------------------|-------------------------|
| Appearance: | Clear colourless liquid |
| Relative density [20° C]: | 1.000 |
| pH-value in use: | 1.95 |
| pH-value neat: | 0.55 |
| Flash point: | N/A |
| Shelf life: | 720 days |
| Scent: | neutral |

The above data is typical of normal production and should not be taken as a specification.

Safe handling and storage information

Full guidance on the handling and disposal of this product is provided in a separate Safety Data Sheet. Store in original container. Avoid extremes of temperature. Reacts with oxidisable substances.

Product Compatibility

Under recommended conditions of use, Diversey Oxivir Excel is suitable for use on most materials commonly encountered.

Environmental Information

The surfactants used in this products are ultimately biodegradable in compliance with requirement of the EU Detergent Regulation, EC 648/2004.

Microbiological data

Efficacy proven by compliance with: EN13727, EN13624, EN14476.

| Claim | Contact time | Dilution | ml / 1L water |
|---------------------|--------------|----------|---------------|
| <i>Bactericidal</i> | 30 sec | 2% | 20 ml |
| | 5 min | 1.5% | 15 ml |
| <i>Yeasticidal</i> | 5 min | 4% | 40 ml |
| <i>Virucidal</i> | 5 min | 2% | 20 ml |